

Cleaning RotoBed®Free in a washing system

RotoBed®Free is approved according to the norm IPX6W (Water rating) and washable in a washing system. Follow the instructions carefully to avoid damaging the bed:

- Unplug the power plug. Remove all fabrics (including mattress and covers) as well as all parts of wood.
- All plug connections must be sealed with a plug cover.
- All plugs and plug covers must be fitted with locking-rings (see section 5.12 in the RotoBed®Free Manual) to prevent water from entering the engines.
- Drive all engines fully down/in (so no engine arms protrude) to prevent limescale and rust formation.
- The electrical parts of the bed and mechanical functions must be tested before using the bed again.

NB: The supplied standard castor for a RotoBed®Free care bed **has not been tested** for cleaning in washing systems. Unless the castors are removed before washing, shorter lifespan for the castors should be calculated. Alternatively, special wheels tested for a washing system can be purchased.

RotoBed® points out that washing in machines will cause the bed to age faster over time.