

Mounting of the mattress


RotoBed®Free comes with a mattress consisting of a main part and a foot part.

The main mattress has 5 buttons on the back. First place the three lower buttons in the grooves at the bottom of the bed. Slide the mattress towards the foot end. Now press the two buttons in the centre of the mattress into the corresponding holes in the middle of the bed.

Similarly, the foot mattress has two buttons, which fit into the two round holes in the bed.

Mounting of side rails

RotoBed®Free can be delivered with side rails made of metal or oak veneer. Regardless of the material, the side rails are mounted and removed in the same way:

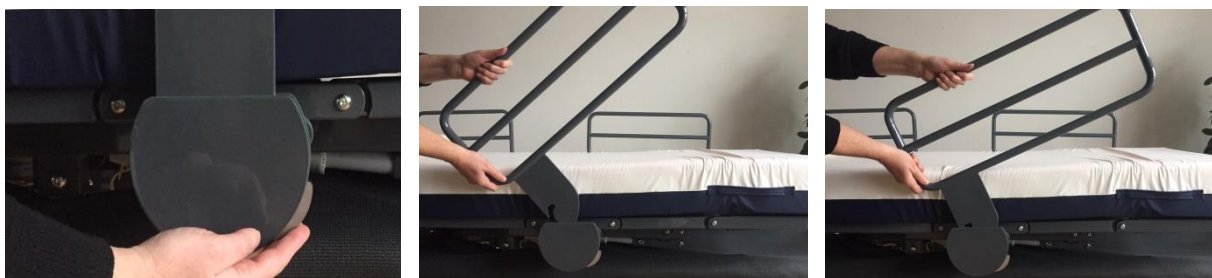

Press the small fitting underneath while lifting the side rail.

The side rails can simply be put back into place and will automatically lock.

NB! The side rails should only be mounted or not mounted. Do not leave them upright and loose in the bracket. This may cause squeeze risk and damage to the bed during rotation.

Castors and brakes



Unlocked castors

As standard, the bed comes with integral Tente Linea double castors.
The castors are rotatable.

The brakes work in pairs and can be operated from both sides of the bed.



Locked castors

Note: RotoBed®Free should not be used by people without an introduction in the use of the bed. The bed is tested according to the latest applicable standards and rules for care beds. RotoBed®Free should be operated exclusively with the remote control. Be sure that neither children nor animals are under the bed when the bed's parts are set. When the bed's leg part is raised, do not overload it with heavy objects or persons. The bed must be used by users within the following criteria: Weight 40-190 kg. Height ≥ 146 cm. BMI ≥ 17 .



Exit: Hold down the **Chair Position** key until the bed stops in an exit position. To facilitate exit you can, by means of the remote control, move the backrest the last bit forward and raise the bed a little in height.



Note: Standard is an angled exit to facilitate the transition to wheelchair or walker. If you wish a perpendicular exit, please see the user manual.




Entry: Get seated as close to the backrest as possible for the best comfort. Hold down the **Sleeping Position** key until the bed is rotated and horizontal. Now, RotoBed®Free works as an ordinary care bed.



Standard seat height without mattress is 32 cm above the floor. For individual settings, see user manual.

Remote control

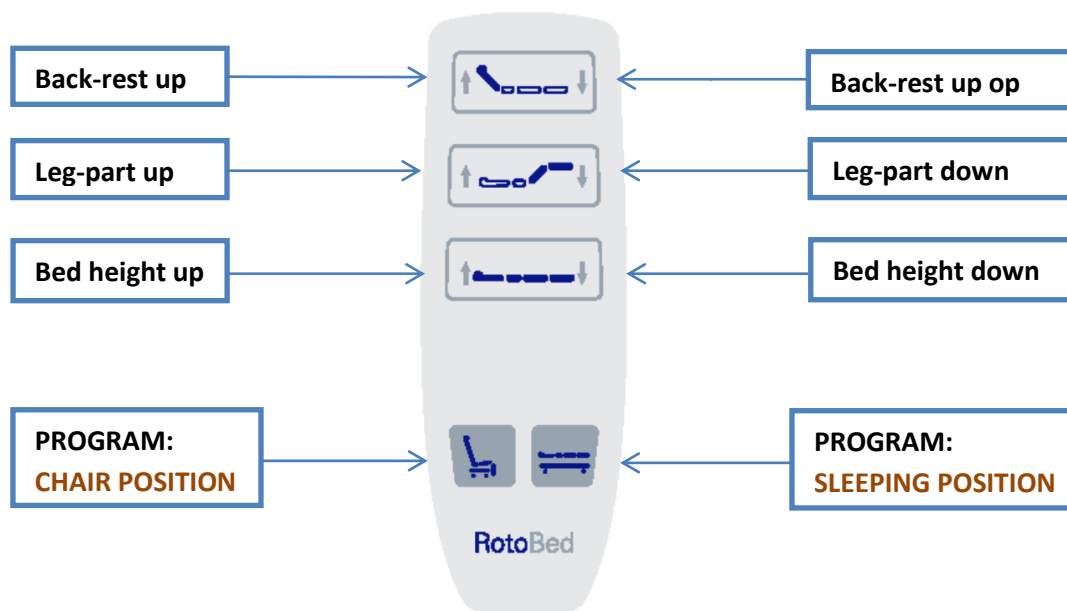

RotoBed®Free can turn either right or left. Therefore, the remote control can also be connected to both the right and left sides of the bed. Optional remote control can be purchased as desired. When the remote control is not in use, place it on the side rail.